Single-use Biopsy Needles

Gebrauchsanweisung beachten | Consult instructions for use | Consulter la notice d'utilisation
 Consultare le istruzioni per l'uso | Consultar as instruções de utilização
 Consultense las instrucciones de uso | Viz návod k použití | Postupujte podľa návodu na použití

Chargenbezeichnung | Batch code | Code du lot | Codice di lotto
 Código do lote | Código de lote | Šarže | Sériový kód

Bestellnummer | Catalogue number | Référence du catalogue | Riferimento di Catalogo
 Número de catálogo | Número de catálogo | Obj. číslo | Číslo objednávky

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en Operating instructions for single-use biopsy needles

Indication for Use:

The Möller Medical Biopsy Needles and Systems are used to obtain core samples and aspiration from tissue by biopsy. The various wall thicknesses, diameters, tip geometries and cutting techniques of the different hollow needle types and biopsy systems affect the quality of the biopsy specimen. Application areas of the products are soft tissue such as kidney, liver, prostate, lymph nodes, breast and bone resp. bone marrow.

Indications for the FNB and MBY needles:

The single-use biopsy needles are exclusively intended for hollow needle biopsies of soft tissue.

Contra-indications:

- This single-use biopsy needle is exclusively for use by authorised, specialised doctors who are familiar with the potential side effects, typical diagnostic findings, restrictions, indications and contra-indications of hollow needle biopsies.
- The use of the product, in particular on patients with blood coagulation disorders or patients treated with anticoagulants, is at the discretion of the attending doctor.

General remarks:

- Prior to use, make sure that the package is intact and check the use-by date. Only use products available in an undamaged package. The package should be opened under sterile conditions. **NOTICE:** Products in an open, damaged or moist package must not be used.
- The product must be kept away from light and stored in a dry and cool place.
- The attending doctor is responsible for the proper use and execution of the examination using this product (these operating instructions only serve as a general guideline).
- This single-use biopsy needle is a sterile single-use product intended for use on one patient. Resterilisation and reuse are not permitted. The reuse of single-use products involves a potential risk of infection for the patient or operator. Contaminated products can cause damage to health, disease or even death of the patient. Cleaning, disinfection or sterilisation can adversely affect essential material properties and product parameters and thus result in product failure.
- The product must be disposed of in accordance with hospital regulations and applicable laws.
- Möller Medical GmbH expressly disclaims any liability for damage caused through improper use of the products or use by unauthorised personnel.
- Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

For USA only: When used for breast biopsy, the product is for diagnosis only. The extent of histological abnormality cannot be reliably determined from its mammographic appearance. Therefore, the extent of removal of the imaged evidence of an abnormality does not predict the extent of removal of a histological abnormality; e.g., malignancy. When the sampled abnormality is not histologically benign, it is essential that the tissue margins be examined for completeness of removal of using standard surgical procedures.

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