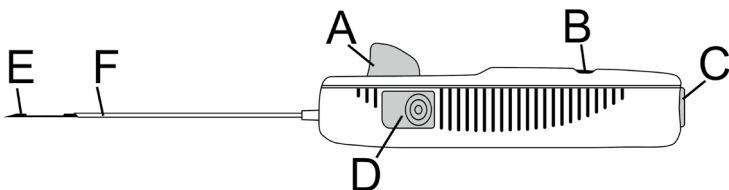
ABG-2020 / ABG-2030 / ABG-2040
Disposable Automatic Biopsy Gun

 Gebrauchsanweisung beachten | Consult instructions for use | Consulter la notice d'utilisation
 Consultare le istruzioni per l'uso | Consultar as instruções de utilização
 Consultense las instrucciones de uso | Viz návod k použití | Postupujte podľa návodu na použitie

 Chargenbezeichnung | Batch code | Code du lot | Codice di lotto
 Código do lote | Código de lote | Sarže | Sériový kód

 Artikelnummer | Catalogue number | Référence du catalogue | Riferimento di Catalogo
 Número de catálogo | Número de catálogo | Obj. číslo | Číslo objednávky

 Vor Nässe schützen | Keep dry | Conserver au sec | Conservare a secco
 Conservare al riparo dalla luce solare | Manter afastado da luz solar | Mantêngase fuera de la luz del sol
 Chraňte před vlhkost | Chráňte pred vlhkosťou

 Vor Lichteinstrahlung schützen | Keep away from sunlight | Conserver à l'abri de la lumière du soleil
 Conservare al riparo dalla luce solare | Manter afastado da luz solar | Mantêngase fuera de la luz del sol
 Chraňte před sluncem | Chráňte pred slnečným žiarením

 Hersteller | Manufacturer | Fabricant | Fabricante
 Fabricante | Fabricante | Výrobce | Výrobca

 Herstellungsdatum [JJJJ-MM-TT] | Date of manufacture [YYYY-MM-DD] | Date of fabrication [AAAA-MM-JJ]
 Data di fabbricazione [AAAA-MM-GG] | Data de fabrico [AAAA-MM-DD] | Fecha de fabricación [AAAA-MM-DD]
 Datum výroby [RRRR-MM-DD] | Dátum výroby [RRRR-MM-DD]

 Verwendbar bis [JJJJ-MM-TT] | Use by [YYYY-MM-DD] | Utiliser jusqu'au [AAAA-MM-JJ]
 Utilizzare entro [AAAA-MM-GG] | Prazo de validade [AAAA-MM-DD] | Fecha de caducidad [AAAA-MM-DD]
 Použitelné do [RRRR-MM-DD] | Spotřebujte do [RRRR-MM-DD]

 Mit Ethylenoxid sterilisiert | Sterilised using ethylene oxide | Méthode de stérilisation utilisant de l'oxyde d'éthylène | Sterilizzato mediante ossido di etilene | Esterilizado com óxido de etileno
 Esterilizado utilizando óxido de etileno | Sterilizováno etylenoxidem | Sterilizácia etylénoxidom

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Indication for Use: The Möller Medical Biopsy Needles and Systems are used to obtain core samples and aspiration from tissue by biopsy. The various wall thicknesses, diameters, tip geometries and cutting techniques of the different hollow needle types and biopsy systems affect the quality of the biopsy specimen. Application areas of the products are soft tissue such as kidney, liver, prostate, lymph nodes, breast and bone resp. bone marrow.

Indications for the ABG: The disposable automatic biopsy guns are intended for biopsies of soft tissue.

Contra-indications:

- This disposable automatic biopsy gun is exclusively for use by authorised, specialised physicians who are familiar with the potential side effects, typical diagnostic findings, restrictions, indications and contra-indications of hollow needle biopsies.
- The use of the product, in particular on patients with a nickel allergy, blood coagulation disorders or patients treated with anticoagulants, is at the discretion of the attending physician.

Instructions for use:

1. Prior to use, make sure that the package is intact and check the use-by date. Only use products available in an undamaged package. The package should be opened under sterile conditions. **NOTICE:** Products in an open, damaged or moist package must not be used.
2. Take the disposable automatic biopsy gun out of the package and then remove the protective cover.
3. Check the stiletto tip (E) and the needle's cutting edge (F) for damage. **NOTICE:** The disposable automatic biopsy gun must not be used if damaged or defective in any way that could adversely affect its intended use or if the sample kerf of the stiletto protrudes when the disposable automatic biopsy gun is loaded.
4. To prevent infection, make sure to work under sterile conditions, including skin disinfection, sterile covering if applicable and sterile gloves.
5. Load the needle by pulling back the central loading lever (A) with your thumb or index finger as far as it will go and then release it to allow it to return to its initial position. In this way the cannula of the disposable automatic biopsy gun is loaded. Repeat this operation a second time, i. e. pulling back the same lever again as far as it will go and then release it to allow it to return to its initial position. In this way the stylet is loaded and a visual sign can be seen in the rear hole (B) indicating that loading is complete.
6. Insert the tip of the needle in the site selected for the biopsy.
7. Continue the procedure by pressing the rear button (C) or the side button (D), whichever is more comfortable for the physician. Then choose either alternative 7.1 or 7.2 for the products ABG-2020 and ABG-2030. Alternative 7.2 applies exclusively to the product ABG-2040:
 - 7.1. Lightly press button (C) or (D) without pushing it fully down so that only the stylet moves forward thus exposing the sampling cavity outside the cannula. Then fully press down the button to let the cannula move forward. In this way, insertion is accomplished with two distinct manual actions and the time between one and the next can be decided by the physician. Only applies to ABG-2020 and ABG-2030.
 - 7.2. Fully press down the button (C) or (D). In this way you can simultaneously move the stylet and the cannula forward with one manual action (the stylet and the cannula move forward at slightly staggered times). Applies to ABG-2020/ABG-2030/ABG-2040.
8. Slowly remove the needle from the patient.
9. To retrieve the fragment contained in the cavity in proximity of the needle tip, push down on the central loading lever (A) as far as it will go and then release it to allow automatic return to its original position. **NOTICE:** Unlike the initial needle loading operation, to retrieve the fragment, the loading lever needs to be pushed only once and not twice.
10. Finally, once the sampling area containing the fragment has been uncovered, scrape the sample taken onto a slide.

General remarks:

- The product must be kept away from light and stored in a dry and cool place.
- The attending physician is responsible for the proper use and execution of the examination using this product (these operating instructions only serve as a general guideline).
- This disposable automatic biopsy gun is a sterile single-use product intended for use on one patient. Resterilisation and reuse are not permitted. The reuse of single-use products involves a potential risk of infection for the patient or operator. Contaminated products can cause damage to health, disease or even death of the patient. Cleaning, disinfection or sterilisation can adversely affect essential material properties and product parameters and thus result in product failure.
- It is also recommended not to use the device if particularly risky manoeuvres are necessary where the needle will be subjected to high torsional or bending forces which might cause it to snap at the height of the grip.
- The product must be disposed of in accordance with hospital regulations and applicable laws.
- Möller Medical GmbH expressly disclaims any liability for damage caused through improper use of the products or use by unauthorised personnel.
- Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

 **For USA only:** When used for breast biopsy, the product is for diagnosis only. The extent of histological abnormality cannot be reliably determined from its mammographic appearance. Therefore, the extent of removal of the imaged evidence of an abnormality does not predict the extent of removal of a histological abnormality; e.g., malignancy. When the sampled abnormality is not histologically benign, it is essential that the tissue margins be examined for completeness of removal of using standard surgical procedures.

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