de

REF Artikelnummer

QTY Menge

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 Herstellungsdatum (JJJJ-MM-TT)

LOT Charge

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 Date of manufacture
YYYY-MM-DD

LOT Batch code

STERILEEO Sterilized using ethylene oxide

 Do not reuse

 Consult instructions for use

 Temperature limitation
-10°C (+14°F) to +50°C (+122°F)

 Do not resterilise

 Do not use if package is damaged

 Keep dry

 Keep away from sunlight

 Manufacturer

RX ONLY

Attention:
Under US Federal law
this device may be
sold only to a
physician or ordered
by a physician.

CE 0482

You can find further information on the symbols used on our homepage:

www.moeller-medical.com/glossary-symbols

TLA Tubing Liposat® Pro plus (REF. 00003997)

Attention: Do not use the product if the packaging is damaged. Safe use only in combination with the following products:

Liposat®		Vibrasat®		Accessories	
Product	REF	Product	REF	Product	REF
Liposat® Pro plus	00003974	Vibrasat® power	00002246	TLA Luer-Lock Adapter	00004027
		Vibrasat® Pro	00003920	Cannulas for Vibrasat®	see brochure
				Sterican Cannulas	00002535
				Multi-way-tap	00002278 / 00002279
				Manual handle	00002254
Table 1		Table 2		Table 3	

Intended use: The TLA Tubing **Liposat® Pro plus** tubing set is used both for medical indications, including those accompanied by a change in fatty tissue, and for aesthetic body shaping.

The TLA Tubing **Liposat® Pro plus** is used to administer tumescent local anaesthesia, other aqueous infusion solutions, as well as endogenous subcutaneous tissue and its components, into the body.

The tubing set, sterilised with ethylene oxide, is marketed as a disposable product and may only be used in conjunction with **Liposat® Pro plus**.

Single use: The re-use of a single-use product presents a potential risk of infection for the patient or operator. Contaminated products can damage health and cause illness or even death of the patient. Cleaning, disinfection and sterilization can adversely affect essential material properties and product parameters and thus result in product failure.

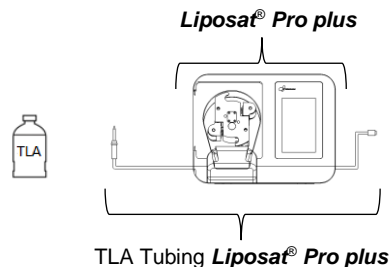
Precautions: Before use, users should be familiar with the **Liposat®** pump and **Vibrasat®**. Follow the instructions for use of the **Liposat®** pumps and **Vibrasat®** (see *Tables 1* and *2*).

Contraindication: See the instructions for use of the **Liposat®** pumps (see *Table 1*).

Quick guide: These instructions are only to be used together with the instructions for use for **Liposat®** pumps (see *Table 1*), **Vibrasat®** (see *Table 2*) and the accessories (see *Table 3*).

- 1) Open the front cover of the pump.
 - 2) The pump head automatically rotates into a suitable position for inserting the tubing set.
 - 3) Insert the pump segment of the tube into the left tube holder until it clicks into place. Make sure the spike connector of the tube is on the left side. Adjust the tube length so that the connection to the saline bag is possible.
 - 4) Hold the tubing in position while pressing and holding the button  for inserting the tubing set. You can also turn the rotor by hand in the direction of the arrows.
 - 5) The rotor rotates and threads the tube. Guide the tube by hand in such a way that the rotor can pick it up optimally.
 - 6) Stop the rotor rotation by releasing the button when the tube is in a suitable position.
 - 7) Insert the end of the tube into the right tube holder.
 - 8) Pierce the spike located on the tubing set into the saline bag.
 - 9) Connect the right side of the tube with one of the infiltration options of *Table 3*.
- The tubing set is completely assembled.
- Note that the spike connector is on the left side and the connector for the infiltration options is on the right side.**

Follow the instructions for use of the Liposat® pumps for handling the infiltration tubing set.

- Sterican cannulas (with multi-way-tap)
- Single TLA infiltration cannula on vibrating handle with TLA Luer-Lock adapter
- Single TLA infiltration cannula on manual handle with TLA Luer-Lock adapter

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Möller Medical GmbH