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30007617 2021-02 C

# en

## Tubing for tumescent pump Liposat® (REF. 00002251)

**Attention:** Do not use the product if the packaging is damaged.  
Safe use only in combination with the following products:

Liposat®		Vibrasat®		Accessories	
Product	REF	Product	REF	Product	REF
<i>Liposat® power</i>	00002274	<i>Vibrasat® power</i>	00002246	TLA Luer-Lock Adapter	00004027
<i>Liposat® Pro</i>	00003977	<i>Vibrasat® Pro</i>	00003920	Cannulas for <i>Vibrasat®</i>	see brochure
				Sterican Cannulas	00002535
				Multi-way-tap	00002278 / 00002279
				Manual handle	00002254
Table 1		Table 2		Table 3	

**Intended use:** The *Liposat® power* tubing set is used both for medical indications, including those accompanied by a change in fatty tissue, and for aesthetic body shaping. The tube set for *Liposat® power* is used to promote aqueous solutions to produce tumescence local anaesthesia as well as body-owning, subcutaneous tissue, and its components into the body. The tubing set, sterilised with ethylene oxide, is marketed as a disposable product and may only be used in conjunction with *Liposat® power* and *Liposat® Pro*.

**Single use:** The reuse of a single-use product presents a potential risk of infection for the patient or operator. Contaminated products can damage health and cause illness or even death of the patient. Cleaning, disinfection, and sterilization can adversely affect essential material properties and product parameters and thus result in product failure.

**Precautions:** Before use, users should be familiar with the *Liposat®* pump and *Vibrasat®*. Follow the instructions for use of the *Liposat®* pumps and *Vibrasat®* (see Tables 1 and 2).

**Contraindication:** See the instructions for use of the *Liposat®* pumps (see Table 1).

**Quick guide:** These instructions are only to be used together with the instructions for use for *Liposat®* pumps (see Table 1), *Vibrasat®* (see Table 2) and the accessories (see Table 3).

### Liposat® power

- 1) Open the front cover of the pump.
- 2) Load the pump tube segment into the pump. The pump tube segment can be found between the adapters, which are intended to be attached into the adapter receptacle of the pump.
- 3) Follow the flow direction arrow on the pump. The pump rotates to the left (counter clockwise). Saline solution must be pumped out of its container towards the patient.
- 4) The pump tube segment is firmly attached to the 1.5-metre long PVC afferent tube with its spike, to draw saline solution out of the container.
- 5) Connect one side of the efferent tube to the tube pump segments Luer connector. Connect the other end of the efferent tube to one of the infiltration ports as shown in table 3.

The tubing set is completely assembled.

**Note that the spike connector is on the right side and the connector for the infiltration options is on the left side.**

### Liposat® Pro

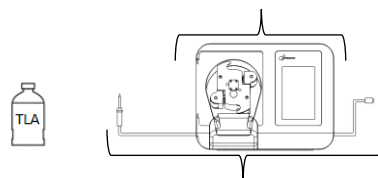
- 1) Open the front cover of the pump.
- 2) The pump head rotates automatically to a position suitable for inserting the tubing set.
- 3) Insert the pump segment of the tube into the left tube holder until it clicks into place. Make sure the spike connector of the tube is on the left side. Adjust the tube length so that the connection to the saline bag is possible.
- 4) Hold the tube in position while pressing and holding the button for inserting the tubing set. You can also turn the rotor by hand in the direction of the arrows.
- 5) The rotor rotates and threads the tube. Guide the tube by hand in such a way that the rotor can pick it up optimally.
- 6) Stop the rotor rotation by releasing the button when the tube is in a suitable position.
- 7) Insert the end of the tube into the right tube holder.
- 8) Pierce the spike located on the tube set into the saline bag.
- 9) Connect the right side of the tube with one of the infiltration options of Table 3.

The tube set is completely assembled.

**Note that the spike connector is on the left side and the connector for the infiltration options is on the right side.**

### Follow the instructions for use of the Liposat® pumps for handling the infiltration tube set.

#### Liposat® power / Pro

Tubing for tumescent pump  
*Liposat®*

- Sterican cannulas (with multi-way-tap)
- Single TLA infiltration cannula on vibrating handle with TLA Luer-Lock adapter
- Single TLA infiltration cannula on manual handle with TLA Luer-Lock adapter

*Vibrasat®*  
or accessories (Table 3)

**REF** Catalog number

**QTY** Quantity

Use by Date  
YYYY-MM-DD

Date of manufacture  
YYYY-MM-DD

**LOT** Batch code

**STERILE EO** Sterilized using ethylene oxide

Do not reuse

Consult instructions for use

Temperature limitation  
-10°C (+14°F) to +50°C (+122°F)

Do not resterilise

Do not use if package is damaged

Keep dry

Keep away from sunlight

Manufacturer

**R ONLY**

Attention:  
Under US Federal law  
this device may be sold  
only to a physician or  
ordered by a physician.

Further information on the symbols used  
can be found on our homepage:  
[www.moeller-medical.com/glossary-symbols](http://www.moeller-medical.com/glossary-symbols)

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