# de

**REF** Artikelnummer

**QTY** Menge

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 Herstellungsdatum (JJJJ-MM-TT)

**LOT** Charge

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Keep away from sunlight

Manufacturer

**R ONLY**  
Attention: Under US Federal law this device may be sold only to a physician or ordered by a physician.

Further information on the symbols used can be found on our homepage: [www.moeller-medical.com/glossary-symbols](http://www.moeller-medical.com/glossary-symbols)

## Tubing for tumescent pump Liposat® basic (REF. 00002492)

**Attention:** Do not use the product if the packaging is damaged. Safe use only in combination with the following products:

Pump		Accessories	
Product	REF	Product	REF
<i>Liposat® basic</i>	00002498	Multi-way-tap	00002278 /00002279
		Manual handle	00002254
		Sterican Cannulas	00002535
		Cannulas for <i>Vibrasat®</i>	see brochure
		TLA Luer-Lock Adapter	00004027

Table 1

Table 2

**Intended use:** The *Liposat® basic* tubing set is used both for medical indications, including those accompanied by a change in fatty tissue, and for aesthetic body shaping. The tube set for *Liposat® basic* is used to promote aqueous solutions to produce tumescence local anaesthesia as well as body-owning, subcutaneous tissue, and its components into the body. The tubing set, sterilised with ethylene oxide, is marketed as a disposable product and may only be used in conjunction with *Liposat® basic*.

**Single use:** The reuse of a single-use product presents a potential risk of infection for the patient or operator. Contaminated products can damage health and cause illness or even death of the patient. Cleaning, disinfection, and sterilization can adversely affect essential material properties and product parameters and thus result in product failure.

**Precautions:** Before use, users should be familiar with the *Liposat® basic* pump. Follow the instructions for use of the *Liposat® basic* pump.

**Contraindication**

See the instructions for use of the *Liposat® basic* pump.


**Quick guide**

These instructions are only to be used together with the instructions for use for the *Liposat® basic* pump and the corresponding accessories.

- 1) Open the front cover of the pump.
  - 2) Insert the tube adapter of the suction side of the pump tube into the pump holder. By turning the pump rotor to the left by hand, the pump tube is threaded over the pump tube guides.
  - 3) The direction of rotation of the pump is anticlockwise. The direction of rotation of the pump, is the delivery direction of the saline solution.
  - 4) Insert the spike in the saline bag.
  - 5) Connect the second end piece of the tube with one of the infiltration options of Table 2.
- The tubing set is completely assembled.

**Note that the spike connector is on the right side and the connector for one of the infiltration options of table 2 is on the left side.**

**Follow the instructions for use of the Liposat® basic pump for handling the infiltration tube set.**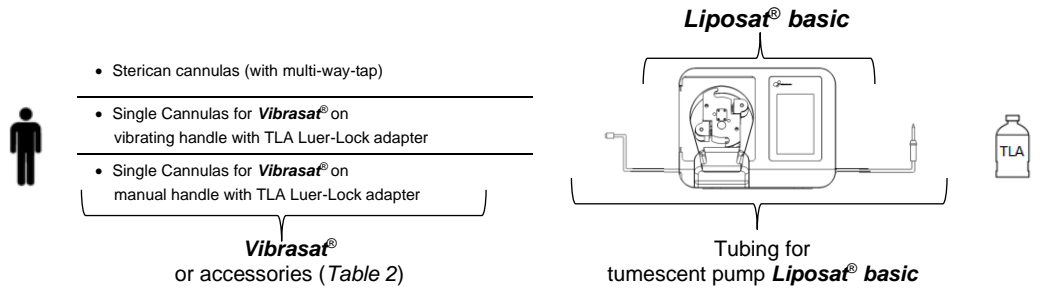

Möller Medical GmbH  
Wasserkuppenstraße 29-31  
36043 Fulda, Germany  
Tel. +49 (0) 661 / 94 19 5 – 0  
Fax. +49 (0) 661 / 94 19 5 – 850  
<http://www.moeller-medical.com>  
[info@moeller-medical.com](mailto:info@moeller-medical.com)

CE 0482 MöllerMedical