

INSTRUCTIONS FOR USE en

Footswitches



IMPORTANT

READ CAREFULLY BEFORE USE

KEEP THESE INSTRUCTIONS FOR FUTURE CONSULTATION

© Möller Medical GmbH
All rights reserved.

No part of this documentation may be reproduced or translated in any way whatsoever without prior written approval from Möller Medical GmbH. The status of the information, specifications and figures in these instructions for use is indicated by the version number on the last page. Möller Medical GmbH reserves the right to make changes in terms of technology, functions, specifications, design and information at any time and without prior notification.

Contents**Contents**

1	General safety instructions	4
1.1	Explanation of the safety symbols used	4
1.1.1	Symbols in the instructions for use.....	4
1.1.2	Symbols appearing on the device	4
1.1.3	Symbols on the retail packaging	4
1.2	Warning notices	6
1.3	Further information.....	6
2	Product description	7
2.1	Intended use	7
2.2	Footswitch.....	7
2.3	1-Pedal Footswitch	7
2.4	3-Pedal Footswitch	8
3	Help in the event of fault	9
4	Disposable.....	10

1 General safety instructions

1.1 Explanation of the safety symbols used

In these instructions for use, important information is indicated visually. These references are prerequisites for preventing hazards to patients and operating personnel, as well as for avoiding damages or malfunctioning of the device.

1.1.1 Symbols in the instructions for use



Caution



Information or help

1.1.2 Symbols appearing on the device



Catalogue number



Batch code



Date of manufacture (YYYY-MM-DD)



Manufacturer



Unique identifier of a medical device



Medical devices



Consult instructions for use

IPX7

Protection against pressured water jets and the ingress of water in case of short-term flooding

IPX8

Protection against pressured water jets and the ingress of water in case of permanently-term flooding










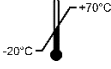


Return and disposal as per the WEEE Directive

1.1.3 Symbols on the retail packaging



Consult instructions for use

General safety instructions

	Caution
	Catalogue number
	Batch code
	Unique identifier of a medical device
	Medical devices
	Date of manufacture (YYYY-MM-DD)
	Manufacturer
	Temperature limit
	Packaging unit
	Return and disposal as per the WEEE Directive

Rx ONLY

Attention: Under US Federal law, this device may be only sold to a physician or ordered by a physician.

Further information on the symbols used can be found on our homepage:

www.moeller-medical.com/glossary-symbols

1.2 Warning notices



- The devices must not be modified.
- No liquids must be allowed to penetrate the voltage-carrying parts of the devices.
- When cleaning, ensure that no cleaning agent runs into the connector sockets.
- Disconnect the power cable before cleaning.
- Replace connecting cables of all kinds even if they are only slightly damaged; make sure not to roll over cables.
- Keep the cables away from heat sources. This prevents the insulation from melting which could cause a fire or an electric shock.
- Do not use force to push plugs into sockets.
- When removing plugs, do not pull on the cables. To remove, release the plug lock if necessary.
- Do not subject the devices to strong heat or fire.
- Do not subject the devices to hard impacts.
- If heat, fumes or smoke appear, disconnect the devices from the mains immediately.

1.3 Further information

Follow the further information and notes in the instructions for use of the **Vibrasat®** and **Liposat®** series.

IFU Liposat® Pro/ Pro plus	93007492
IFU Liposat® power	20007191
IFU Vibrasat® Pro	93007103
IFU Vibrasat® power	92007181

You find all instruction for use on our homepage:

www.moeller-medical.com/ifu-ga

Product description

2 Product description

2.1 Intended use

The footswitches can be used to remotely control the devices of the **Vibrasat®** and **Liposat®** series.

2.2 Footswitch

Two footswitches can be connected to the devices at the same time (applicable for Pro devices). Both connection sockets on the rear panel have the same priority and the footswitches behave identically. The connection sockets for the footswitches have coloured ring markings. If present, attach the matching coloured cable markers to the footswitch cables for reliable identification of the correct connection.



When plugging in the footswitch, make sure that the groove on the plug points upwards and thus fits into the connection socket. A twisted installation will lead to malfunction of the device.

You can use the additional cable markings to colour-code the footswitches of your other Möller Medical devices to identify them more easily.

The footswitches have the same setting options as the buttons on the devices of the **Vibrasat®** and **Liposat®** series.

2.3 1-Pedal Footswitch



Figure 1: 1-Pedal Footswitch

By pressing the pedal of the footswitch, the **Liposat®** and **Vibrasat®** series devices can be switched on and off.

Footswitches

Product description

The 1-Pedal Footswitch can be ordered as follows:

Electrical Footswitch

(2 m wire)

Catalogue no.: 93003545

Electrical Footswitch

(5 m wire)

Catalogue no.: 00003982

2.4 3-Pedal Footswitch



Down pedal On/Off pedal Up pedal

Figure 2: 3-Pedal Footswitch

The **Liposat**[®] and **Vibrasat**[®] series devices can be switched on and off by pressing the On/Off pedal on the footswitch. The Up and Down pedal on the footswitch can be used to set the vibration speed or the flow rate.

The 3-Pedal Footswitch can be ordered as follows:

3-Pedal Footswitch

(2 m wire)

Catalogue no.: 93003517

3-Pedal Footswitch

(5 m wire)

Catalogue no.: 00003981

Help in the event of fault

3 Help in the event of fault

Footswitch doesn't react	The connection cable of the footswitch is not connected or has been plugged in incorrectly. Check the plug connection.
If all the specified actions do not lead to success, contact the service centre of the Möller Medical GmbH.	

Möller Medical GmbH service centre:

Möller Medical GmbH
 Wasserkuppenstrasse 29-31
 36043 Fulda, Germany



Tel. +49 (0) 661 / 94 19 5 – 0
 Fax +49 (0) 661 / 94 19 5 – 850
www.moeller-medical.com
info@moeller-medical.com

Service

Tel: +49 (0) 661 94195 - 108
 Fax: +49 (0) 661 94195 - 850
 E-Mail: service@moeller-medical.com

4 Disposable



This device includes materials which must be disposed of in an environmentally friendly manner. The European Directive 2012/19/EU on waste electrical and electronic equipment (WEEE2) applies to these devices. They thus bear the symbol with a crossed-out bin on the rating plate.

Return devices which are no longer used to Möller Medical GmbH. This ensures that the devices are disposed of in compliance with the national requirements of the WEEE Directive.



Catalogue number of
instructions for use
(REF) 93007903



Möller Medical GmbH
Wasserkuppenstrasse 29-31
36043 Fulda, Germany



Tel. +49 (0) 661 / 94 19 5 – 0
Fax +49 (0) 661 / 94 19 5 – 850
www.moeller-medical.com
info@moeller-medical.com