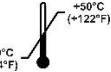
# de

## TLA Luer-Lock Adapter (REF. 00004027) Kurzanleitung

### Achtung

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## TLA Luer-Lock Adapter (REF. 00004027) Short Guide

### Caution

Do not use the product if the packaging is damaged.  
Safe to use with the following products:

<i>Liposat</i> <sup>®</sup>		<i>Vibrasat</i> <sup>®</sup>		<i>Tubing sets Liposat</i> <sup>®</sup>	
product name	REF	product name	REF	product name	REF
<i>Liposat</i> <sup>®</sup> power	00002274	<i>Vibrasat</i> <sup>®</sup> power	00002246	Tubing for tumescent pump <i>Liposat</i> <sup>®</sup> power	00002251
<i>Liposat</i> <sup>®</sup> Pro	00003977	<i>Vibrasat</i> <sup>®</sup> Pro	00003920	TLA Tubing <i>Liposat</i> <sup>®</sup> Pro plus	00003997
<i>Liposat</i> <sup>®</sup> Pro plus	00003974				
Table 1		Table 2		Table 3	

**Intended use:** The TLA-Luer-Lock Adapter is used in aesthetic body contouring and in general medical indications to connect the Möller Medical Tube sets with Luer-Lock for Liposat pumps with the Vibrasat liposuction cannulas. The TLA-Luer-Lock Adapter is sterilised with ethylene oxide and sold as a disposable product.

**Single use:** Reusing disposable products poses a potential risk of infection for patients or operators. Contaminated products can damage health and cause illness or even death of the patient. Cleaning, disinfection or sterilisation can adversely affect essential material properties and product parameters and thus result in product failure.

**Precautions:** Users should be familiar with the *Liposat*<sup>®</sup> pump and *Vibrasat*<sup>®</sup> before use. For this reason, please also refer to the Instructions for use of the *Liposat*<sup>®</sup> pumps and *Vibrasat*<sup>®</sup> (see tables 1 and 2).

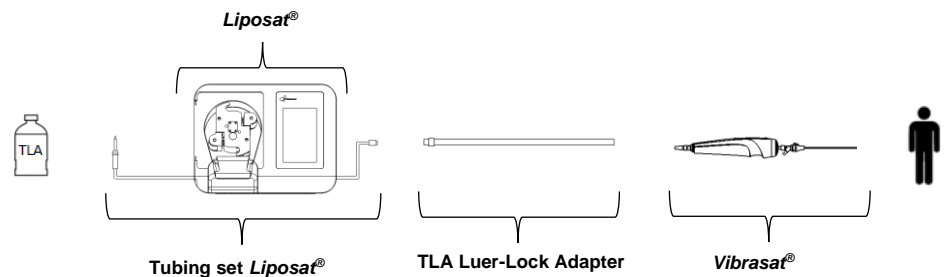
**Contraindications:** See Instructions for use *Liposat*<sup>®</sup> pumps (see Table 1).

### Quick guide

This Instruction for Use is only valid together with the Instructions for use for *Liposat*<sup>®</sup> pumps (see Table 1), *Vibrasat*<sup>®</sup> (see Table 2) and the tubing sets for *Liposat*<sup>®</sup> (see Table 3).

- 1) Remove tubing set from packaging. Pay attention to hygienic standards.
- 2) Pass the open end of the **TLA Luer-Lock Adapter** through the tube holder on the *Vibrasat*<sup>®</sup> handle.
- 3) Now push the open end onto the cannula holder or the tube connector of the cannula.
- 4) Connect the female Luer-Lock of the **TLA Luer-Lock Adapter** to the male Luer-Lock of the tubing set for *Liposat*<sup>®</sup> (see Table 3).

Follow the Instructions For Use of the *Liposat*<sup>®</sup> pumps for handling the infiltration tubing set.

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Keep away from  
sunlight

Manufacturer

Attention: Under  
US federal law, this  
device may be only  
sold to a physician  
or ordered by a  
physician.

**Rx ONLY**

**CE 0482**

For further information about  
symbols used please refer to our  
homepage: [www.moeller-medical.com/glossary-symbols](http://www.moeller-medical.com/glossary-symbols)