COX-1010, COX-1020, COX-1030, COX-1040, COX-2000 und COX-2020 Coaxial Needle

Gebrauchsanweisung beachten | Consult instructions for use | Consulter la notice d'utilisation
 Consultare le istruzioni per l'uso | Consultar as instruções de utilização
 Consultense las instrucciones de uso | Viz návod k použití | Postupujte podľa návodu na použitie

Chargenbezeichnung | Batch code | Code du lot | Codice di lotto
 Código do lote | Código de lote | Šarže | Sériový kód

Bestellnummer | Catalogue number | Référence du catalogue | Riferimento di Catalogo
 Número de catálogo | Número de catálogo | Obj. číslo | Číslo objednávky

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COX-1010, COX-1020, COX-1030, COX-1040, COX-2000 and COX-2020 coaxial needles

Indication for Use:

The Möller Medical Biopsy Needles and Systems are used to obtain core samples and aspiration from tissue by biopsy. The various wall thicknesses, diameters, tip geometries and cutting techniques of the different hollow needle types and biopsy systems affect the quality of the biopsy specimen. Application areas of the products are soft tissue such as kidney, liver, prostate, lymph nodes, breast and bone resp. bone marrow.

Indications for the COX needles:

For safe needle guidance and for taking several biopsy specimens with only one single puncture.

Contra-indications:

- These COX coaxial needles are exclusively for use by authorised, specialised doctors who are familiar with the potential side effects, typical diagnostic findings, restrictions, indications and contra-indications of the use of these COX coaxial needles.
- The use of the product, in particular on patients with blood coagulation disorders or patients treated with anti-coagulants, is at the discretion of the attending doctor.

Compatibility:

The examples in the list below illustrate compatibility with the single-use biopsy needles.

Coaxial needle	Single-use biopsy needle	Coaxial needle	Single-use biopsy needle
COX-1010	DNG-1010	COX-2000	SBG-2010
COX-1020	DNG-1020	COX-2000	ABG-2020
COX-1030	DNG-1030	COX-2020	DNG-2020
COX-1040	DNG-1040		

Thanks to the extension adapters in a uniform colour design, the COX coaxial needles can be conveniently assigned to the correct Gauge sizes of the corresponding single-use biopsy needles for each individual use.

Preparatory measures:

- Prior to use, make sure that the package is intact and check the use-by date. Only use products available in an undamaged package. The package should be opened under sterile conditions. **NOTICE:** Products in an open, damaged or moist package must not be used.
- Take the needle out of the package and then remove the protective cover.
- Check the COX coaxial needle's cutting edge for damage and make sure that the longer single-use biopsy needle to be inserted slides freely within the COX coaxial needle. **NOTICE:** The COX coaxial needle must not be used in case of damage or any other defect which can adversely affect its intended use.

Special feature:

The COX coaxial needles are equipped with infinitely variable springs which enable the precise definition of the puncture depth.

General remarks:

- The product must be kept away from light and stored in a dry and cool place.
- The attending doctor is responsible for the proper use and execution of the examination using this product (these operating instructions only serve as a general guideline).
- These COX coaxial needles are a sterile single-use product intended for use on one patient. Resterilisation and reuse are not permitted. The reuse of single-use products involves a potential risk of infection for the patient or operator. Contaminated products can cause damage to health, disease or even death of the patient. Cleaning, disinfection or sterilisation can adversely affect essential material properties and product parameters and thus result in product failure.
- The product must be disposed of in accordance with hospital regulations and applicable laws.
- Möller Medical GmbH expressly disclaims any liability for damage caused through improper use of the products or use by unauthorised personnel.
- Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

For USA only: When used for breast biopsy, the product is for diagnosis only. The extent of histological abnormality cannot be reliably determined from its mammographic appearance. Therefore, the extent of removal of the imaged evidence of an abnormality does not predict the extent of removal of a histological abnormality; e.g., malignancy. When the sampled abnormality is not histologically benign, it is essential that the tissue margins be examined for completeness of removal of using standard surgical procedures.

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