
BOW-1070 Bone Marrow Aspiration Biopsy

Gebrauchsanweisung beachten | Consult instructions for use | Consulter la notice d'utilisation
 Consultare le istruzioni per l'uso | Consultar as instruções de utilização
 Consultense las instrucciones de uso | Viz návod k použití | Postupujte podľa návodu na použití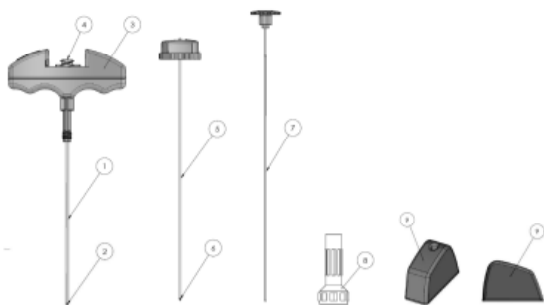

Chargenbezeichnung | Batch code | Code du lot | Codice di lotto
 Código do lote | Código de lote | Sarže | Sériový kód

Bestellnummer | Catalogue number | Référence du catalogue | Riferimento di Catalogo
 Número de catálogo | Número de catálogo | Obj. číslo | Číslo objednávky

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en Operating instructions for BOW-1070 bone marrow aspiration biopsy

Indication for Use: The Möller Medical Biopsy Needles and Systems are used to obtain core samples and aspiration from tissue by biopsy. The various wall thicknesses, diameters, tip geometries and cutting techniques of the different hollow needle types and biopsy systems affect the quality of the biopsy specimen. Application areas of the products are soft tissue such as kidney, liver, prostate, lymph nodes, breast and bone resp. bone marrow.

Indications for the BOW-1070: The BOW-1070 bone marrow aspiration biopsy set is intended for aspiration in the area of the sternum or iliac crest.

Contra-indications:

- This needle set is exclusively for use by authorised, specialised doctors who are familiar with the potential side effects, typical diagnostic findings, restrictions, indications and contra-indications of bone marrow aspiration biopsies.
- The use of the product, in particular on patients with blood coagulation disorders or patients treated with anticoagulants, is at the discretion of the attending doctor.

Preparatory measures:

- Prior to use, make sure that the package is intact and check the use-by date. Only use products available in an undamaged package. The package should be opened under sterile conditions. **NOTICE:** Products in an open, damaged or moist package must not be used.
- Take the products out of the package and then remove the protective covers.
- Check the mandrin tip (6) and the needle's cutting edge (2) for damage and make sure that the mandrin (5) slides freely within the biopsy needle (1). **NOTICE:** The needle sets must not be used in case of damage or any other defect which can adversely affect their intended use.
- **NOTE:** The arrow on the mandrin cap (7) and the cutting edge of the mandrin tip, and also the arrow on the handle (3) and the cutting edge of the biopsy needle, point into the same direction!

Intended use:

1. Correct patient positioning.
2. Definition of the entry point and marking if required.
3. To prevent infection, make sure that you work under sterile conditions, including skin disinfection, sterile covering and sterile gloves.
4. Local anaesthesia of the puncture point and the particularly sensitive periosteum.
5. If required, perform a stab incision using the scalpel in order to facilitate skin penetration.
6. Hold the biopsy needle by the handle, place your index finger on the needle and extend the finger towards the distal end. This enables safe handling and control over the maximum penetration depth (steplessly variable through the spacer ring (8), e.g. for sternal application; removal of the spacer ring, e.g. for iliac crest puncture).
7. Using the biopsy needle, exert rotating pressure in order to form a puncture channel which first extends to the periosteum (resistance is noticeable), and then continue with the utmost care until you have overcome the compacta (the resistance suddenly decreases).
8. After entering the bone marrow cavity, remove the mandrin from the biopsy needle by rotating the locking mechanism 90°.
9. Adaptation of a syringe by means of a luer lock fitting (4) on the biopsy needle.
10. Take an aspiration specimen by exerting negative pressure (withdrawing the syringe plunger).
11. Remove the biopsy needle including the syringe from the puncture channel.
12. Remove the syringe with the aspiration specimen from the biopsy needle.

General remarks:

- The product must be kept away from light and stored in a dry and cool place.
- This needle set is a sterile single-use product intended for use on one patient. Resterilisation and reuse are not permitted. The reuse of single-use products involves a potential risk of infection for the patient or operator. Contaminated products can cause damage to health, disease or even death of the patient. Cleaning, disinfection or sterilisation can adversely affect essential material properties and product parameters and thus result in product failure.
- The attending doctor is responsible for the proper use and execution of the examination using this product (these operating instructions only serve as a general guideline).
- Prior to disposal, carefully place the needle protectors (9) onto the needle tips. To do this, position the needle protectors on a solid and level surface. Hold the needles by the proximal end and then push the distal end into the needle protector hole. To prevent inadvertent punctures, make sure that you do not hold the needle protector with your free hand.
- The product must be disposed of in accordance with hospital regulations and applicable laws.
- Möller Medical GmbH expressly disclaims any liability for damage caused through improper use of the products or use by unauthorised personnel.
- Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

For USA only: When used for breast biopsy, the product is for diagnosis only. The extent of histological abnormality cannot be reliably determined from its mammographic appearance. Therefore, the extent of removal of the imaged evidence of an abnormality does not predict the extent of removal of a histological abnormality; e.g., malignancy. When the sampled abnormality is not histologically benign, it is essential that the tissue margins be examined for completeness of removal of using standard surgical procedures.

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