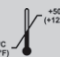
MöllerMedical

Möller Medical GmbH
Wasserkuppenstraße 29-31
36043 Fulda, Germany
www.liquoguard.com

Drainage Set *LiquoGuard*[®]

- de Gebrauchsanweisung
- en Instruction leaflet
- fr Notice d'instructions
- it Foglietto di istruzioni
- sv Anvisningar
- es Folleto de instrucciones
- ru інструкції

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CAUTION:

- Safe to use with the CSF drainage pump *LiquoGuard*[®].
- Do not use the product if the packaging is damaged.
- Observe MRI instructions when using the products with MRI.
- Always observe the instructions on the *LiquoGuard*[®] display as well as the *LiquoGuard*[®] instructions for use (IFU).
- Before the tubing set is removed from the pump or replaced, always close the four-way stopcock between the tubing set and the catheter first.
- Reusing disposable products poses a potential risk of infection for patients or operators. Contaminated products can damage health and cause illness or even death of the patient. Cleaning, disinfection or sterilisation can adversely affect essential material properties and product parameters and thus result in product failure.
- Ensure additional fixation and strain relief of the pressure sensor, the tubes and the catheter between the patient and the pressure sensor, especially for mobile patients or children.

Short drainage guide:

- 1) Remove tubing set from packaging. Pay attention to hygienic standards. Switch on *LiquoGuard*[®].
- 2) Attach adhesive electrode to pressure sensor housing.
- 3) Connect drainage bag to tubing set and hang it onto the holder on the side of the *LiquoGuard*[®].
- 4) Connect the tubing set cable to the running *LiquoGuard*[®] device.
- 5) Follow the menu instructions.
- 6) Insert the tubing set into the pump.
 - a) Open the front panel of the pump.
 - b) Insert Adapter on the right hand of the pump.
 - c) Load the tube set into the rotor by turning the rotor counter clockwise.
 - d) Insert Adapter on the left hand of the pump.

Ensure that the connectors fit and are correctly positioned in the guides to the left and right at the bottom of the pump module. Check that the tube has been correctly inserted.
- 7) Connect the *LiquoGuard*[®] drainage set to the catheter. Open the four-way stopcock between the patient (drainage catheter) and *LiquoGuard*[®]. Open the tube set (not to the atmosphere!!). CSF must be able to flow from the patient to the device.
- 8) Attach the pressure sensor housing to the patient at the height of Foramen Monroi, using the adhesive electrode.
- 9) Vent the tubing set until the pressure sensor by rotating the rotor counter clockwise .
- 10) Close the front panel of the pump.
- 11) Check the *LiquoGuard*[®] display:
 - a) Slight head movements of the patient create pressure changes, see display (ensures communicating liquid columns).
 - b) CSF pulsation of the patient is indicated by pulsation of the pressure curve (small amplitudes for lumbar pressure measurement).
- 12) Check the device settings.
- 13) Always make sure that all valves are set correctly (especially between the tube set and drainage bag).

REF Catalogue number

QTY Unit quantity in packaging

 Total length

 Use by YYYY-MM-DD

 Date of manufacture (YYYY-MM-DD)

LOT Batch code

STERILE Sterilised using ethylene oxide

 Do not reuse

 Consult instruction manual

 Temperature limitation
-10°C (+14°F) to +50°C (+122°F)

 Not suitable for use with MRI

 Suitable for use with MRI
MR Max 3 Tesla Tube Set only

 Do not re-sterilise

 Do not use if packaging is damaged

 Keep dry

 Keep away from sunlight

 Manufacturer

Rx ONLY Attention: Under US Federal law, this device may be only sold to a physician or ordered by a physician.

Short guide for use in an MRI environment:

Leave the tubing set attached to the catheter during MRI to reduce the risk of infection. Avoid unnecessary disconnection or opening of the tubing set against atmosphere.

Do not leave the **LiquoGuard®** monitor connected to the **LiquoGuard®** tubing set during MRI.

Close the **LiquoGuard®** tubing set during MRI. No intracranial pressure reading during MRI.

CAUTION:

- **LiquoGuard®** devices (monitors) may not be used in an MRI environment.
- **LiquoGuard®** tubing sets indicated in the table as MRI compatible may be used in an MRI environment.
- Observe the maximum magnetic field strength limit of 3 tesla.
- No CSF drainage, infiltration or pressure measurements during MRI imaging.
- Tubing sets may remain connected to the catheter (lumbar or ventricular) during MRI, but the four-way stopcock between the tubing set and the catheter must remain closed in order to avoid unintentional drainage, for example.

MR imaging scan

Tubing sets may not be connected to the **LiquoGuard®** device (monitor) during MRI scan.

LiquoGuard® devices (monitors) must be located outside the MRI room. They may remain switched on so the settings are not changed (the maximum permissible time that the tubing set may be separated from the device is 1 hour), but may not be exposed at any time to the MRI's magnetic field.

- 1) Press Standby key for a few seconds and select the *Interrupt* button. Follow the instructions on the display.
- 2) Disconnect the sensor cable from the device.
- 3) Close the four-way stopcock between the patient and the **LiquoGuard®** tubing set (protection of patient).
- 4) Open the front panel on the pump housing.
- 5) Remove adapter on the right hand out of the pump.
- 6) Turn rotor counter clockwise.
- 7) Turn until the tubing set is completely released out of the rotor.
- 8) Remove Adapter on the left hand out of the pump.
- 9) Remove the CSF drainage bag from the bag holder but do not disconnect it from the tubing set. Do not close the valve.
- 10) Lay the tubing set and the drainage bag next to the patient. Do not separate the tubing set from the drainage catheter.
- 11) Disconnect the pressure sensor from the adhesive electrodes.
- 12) Remove the adhesive electrodes from the patient.
- 13) Reconnect the tubing set within 1 hour.

Avoiding image artefacts in MRI

- Do not attach the pressure sensor housing to the patient.
- Do not place the tubing set and the CSF drainage bag on, under or directly next to the patient.
- Remove the adhesive electrode used for attaching the pressure sensor housing to the patient.

After MRI imaging

- 1) Attach adhesive electrode to pressure sensor housing.
 - 2) Connect drainage bag to tubing set and hang it on the holder on the side of the **LiquoGuard®**.
 - 3) Reconnect the tubing set cable to the running **LiquoGuard®** device.
 - 4) Insert the tubing set into the pump.
 - a) Open the front panel of the pump.
 - b) Insert Adapter on the right side.
 - c) Turn rotor counter clockwise to insert the tube set.
 - d) Insert Adapter on the left side.
 - 5) Attach the pressure sensor housing to the patient at the height of Foramen Monroi, using the adhesive electrode.
 - 6) Vent the tubing set until the pressure sensor by rotating the rotor counter clockwise.
 - 7) Close the front panel of the pump.
 - 8) Check the **LiquoGuard®** display:
 - a) Slight head movements of the patient create pressure changes, see display (ensures communicating liquid columns).
 - b) CSF pulsation of the patient is indicated by a pulsation of the pressure curve (small amplitudes for lumbar pressure measurement).
 - 9) Check the device settings.
 - 10) Always make sure that all valves are set correctly (especially between the tube and drainage bag).
- Ensure that the connectors fit and are correctly positioned into the guides to the left and right at the bottom of the pump module. Check that the tube has been correctly inserted.
- Ensure that drainage catheter is connected to **LiquoGuard®** tube set. Open the four-way stopcock between the patient (drainage catheter) and the **LiquoGuard®** (not to the atmosphere!!). CSF must be able to flow from the patient to the device.

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