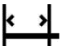
de

Integra Adapterkabel *LiquoGuard*^{®7}

REF 00003854

 4000 mm

REF Artikelnummer

 Gesamtlänge

LOT Fertigungslosnummer, Charge

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 Temperaturbegrenzung
-10°C (+14°F) +50°C (+122°F)

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 Hersteller

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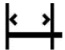
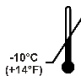
 Keep away from sunlight

 Manufacturer

 Return and disposal are carried out according to the WEEE Directive

 Application part BF

 Caution

 Not suitable for use with MRI

 Date of manufacture (YYYY-MM-DD)

 Humidity, limitation

Rx ONLY

Attention: Under US Federal law this device may be sold only to a physician or ordered by a physician.

For further information about symbols used please refer to our homepage: www.moeller-medical.com/glossary-symbols

Connection

REF 00003500

REF 00003854

Integra
Camino[®]Flex
Ventricular

REF VTUN - Catheter

Always first connect the adapter cable to the pressure sensor. Establish the connection to the *LiquoGuard*[®]7 device (monitor) afterwards.

This sequence is of great importance in order to establish an accurate pressure measurement.

- Only use this medical device in combination with *LiquoGuard*[®]7 REF 00003500
- Refer to IFU *LiquoGuard*[®]7 REF 00003500, chapter “Connecting the Parenchymal- / Tip sensor”
- Non-sterile medical product!
- This product may only be used by medical staff who know and understand all aspects of indication, counter indication, restriction, limitations, typical findings and adverse effects of the medical product and treatment.

Cleaning

All isolating parts of the cable need to be cleaned at least before and after every usage. Use lukewarm soap water and flush proper afterwards. Make sure to protect the electrical connectors in order to avoid malfunction and to protect them from getting wet and from water to intrude into the cable.

Reprocessing instruction

Reprocessed the cable according to the following instructions as needed.

- 1) Prepare a tank of appropriate disinfectant. E.g. use 0,5% HEXANIOS G+R.
- 2) Dip the cable into the solution but ensure to protect the electrical connectors in order to avoid malfunction and to protect them from getting wet and from water to intrude into the cable
- 3) The duration of dwell is 15 minutes
- 4) Flush the cable proper afterwards. Use sterile or low-germ water. Protect the electrical connectors in order to avoid malfunction and to protect them from getting wet and from water to intrude into the cable
- 5) Dry the cable by using absorbent paper
- 6) While the cable is dunked, clean the connectors and cable endings by using an appropriate disinfectant. E.g. use LINGET ANIOS
- 7) Exchange the disinfectant solution after 48 hours max.

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Möller Medical GmbH