

# en

## LiquoGuard®7 Multi-Hub

**REF** Catalog number

**REF** 00003817

Total length

**LOT** Batch number

Consult instructions for use

Temperature limit  
+50°C (+122°F)  
-10°C (+14°F)

Keep dry

Keep away from sunlight

Manufacturer

Return and disposal are carried out according to the WEEE Directive

Application part CF

Caution

Not suitable for use with MRI

Date of manufacture (YYYY-MM-DD)

Humidity, limitation

Re-usable

Blood pressure, invasive measurement

**Rx ONLY**

Attention: Under US Federal law this device may be sold only to a physician or ordered by a physician.

For further information about symbols used please refer to our homepage: [www.moeller-medical.com/glossary-symbols](http://www.moeller-medical.com/glossary-symbols)

### Connection



REF 00003500



REF 93006492



REF 00003817



REF 00003174

Compatible with:  
TruWave disposable pressure transducers (Edwards Lifesciences) e.g. T100209A



- Only use this medical device in combination with LiquoGuard®7 REF 00003500
- It is of great importance to ensure the same height of the arterial blood pressure transducer (MAP) and CSF transducer at all times (refer to IFU LiquoGuard®7 REF 00003500, chapter "Positioning the MAP sensor")
- Non-sterile medical product!
- This product may only be used by medical staff who know and understand all aspects of indication, counter indication, restriction, limitations, typical findings and adverse effects of the medical product and treatment

### Reprocessing instruction

All cleaners detailed in the LiquoGuard®7 REF 00003500 IFU, chapter "Cleaning and Disinfection" may be used. Möller Medical GmbH recommends Isopropanol 70%. Always ensure to protect the cable heads to prevent electrical error.



Möller Medical GmbH  
Wasserkuppenstraße 29-31  
36043 Fulda, Germany



30006568 2020-06 B