
DNG-1010, DNG-1020, DNG-1030, DNG-1040 Single-use Needle f. Reusable Biopsy Gun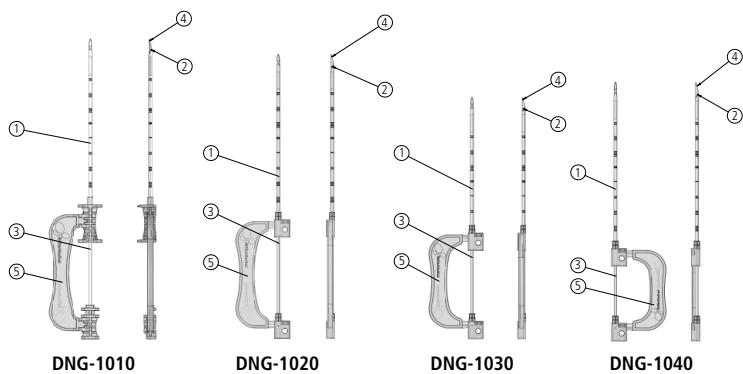

Gebrauchsanweisung beachten | Consult instructions for use | Consulter la notice d'utilisation
 Consultare le istruzioni per l'uso | Consultar as instruções de utilização
 Consultense las instrucciones de uso | Viz návod k použití | Postupujte podľa návodu na použití

Chargenbezeichnung | Batch code | Code du lot | Codice di lotto
 Código do lote | Código de lote | Šarže | Sériový kód

Bestellnummer | Catalogue number | Référence du catalogue | Riferimento di Catalogo
 Número de catálogo | Número de catálogo | Obj. číslo | Číslo objednávky

Vor Nässe schützen | Keep dry | Conserver au sec | Conservare a secco
 Conservar al riparo dalla luce solare | Manter afastado da luz solar | Manténgase fuera de la luz del sol
 Chraňte před vlhkem | Chráňte pred vlhkosťou

Vor Lichteinstrahlung schützen | Keep away from sunlight | Conserver à l'abri de la lumière du soleil
 Conservare al riparo dalla luce solare | Manter afastado da luz solar | Manténgase fuera de la luz del sol
 Chraňte před sluncem | Chráňte pred slnečným žiarením

Hersteller | Manufacturer | Fabricant | Fabricante
 Fabricante | Fabricante | Výrobce | Výrobca

Herstellungsdatum [JJJJ-MM-TT] | Date of manufacture [YYYY-MM-DD] | Date de fabrication [AAAA-MM-JJ]
 Data di fabbricazione [AAAA-MM-GG] | Data de fabrico [AAAA-MM-DD] | Fecha de fabricación [AAAA-MM-DD]
 Datum výroby [RRRR-MM-DD] | Dátum výroby [RRRR-MM-DD]

Verwendbar bis [JJJJ-MM-TT] | Use by [YYYY-MM-DD] | Utiliser jusqu'au [AAAA-MM-JJ]
 Utilizzare entro [AAAA-MM-GG] | Prazo de validade [AAAA-MM-DD] | Fecha de caducidad [AAAA-MM-DD]
 Použitelné do [RRRR-MM-DD] | Spotřebujte do [RRRR-MM-DD]

Mit Ethylenoxid sterilisiert | Sterilized using ethylene oxide | Méthode de stérilisation utilisant de l'oxyde d'éthylène | Sterilizzato mediante ossido di etilene | Esterilizado com óxido de etileno
 Esterilizado utilizando óxido de etileno | Sterilizováno etylenoxidem | Sterilizácia etylénoxidom

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en Operating instructions for single-use biopsy needles for use with biopsy guns

Indication for Use:

The Möller Medical Biopsy Needles and Systems are used to obtain core samples and aspiration from tissue by biopsy. The various wall thicknesses, diameters, tip geometries and cutting techniques of the different hollow needle types and biopsy systems affect the quality of the biopsy specimen. Application areas of the products are soft tissue such as kidney, liver, prostate, lymph nodes, breast and bone resp. bone marrow.

Indications for the DNG-10XX needles: The single-use biopsy needles are exclusively intended for hollow needle biopsies of soft tissue and are combined with biopsy guns – as described in the COMPATIBILITY chapter.

Compatibility: Single-use biopsy needle:	Suitable for Biopsy gun:	Compatible with Coaxial needle:
DNG-1010	ProMag™ 2.2	COX-1010
DNG-1020	ProMag™ Ultra	COX-1020
DNG-1030	ProMag™ Ultra ST	COX-1030
DNG-1040	Bard®Magnum®	COX-1040

Contra-indications:

- This single-use biopsy needle with the corresponding biopsy gun is exclusively for use by authorised, specialised doctors who are familiar with the potential side effects, typical diagnostic findings, restrictions, indications and contra-indications of hollow needle biopsies.
- The use of the product, in particular on patients with blood coagulation disorders or patients treated with anticoagulants, is at the discretion of the attending doctor.

Instructions for use:

- Prior to use, make sure that the package is intact and check the use-by date. Only use products available in an undamaged package. The package should be opened under sterile conditions.
NOTICE: Products in an open, damaged or moist package must not be used.
- Take the single-use biopsy needle (1) incl. its stiletto (3) (delivered ready for use, with spacer (5) and assembled) out of the package and then remove the protective cover.
- Check the stiletto tip (4) and the needle's cutting edge (2) for damage. **NOTICE:** The single-use biopsy needle must not be used in case of damage or any other defect which can adversely affect its intended use.
- Insert the single-use biopsy needle in accordance with the biopsy gun manufacturer's guidelines and operating instructions. **NOTICE:** The single-use biopsy needle must always be inserted in the biopsy gun using a spacer. If the sample kerf of the stiletto protrudes when the biopsy gun is tightened, the single-use biopsy needle must not be used. The spacer must be removed before releasing the gun.

General remarks:

- The product must be kept away from light and stored in a dry and cool place.
- The attending doctor is responsible for the proper use and execution of the examination using this product (these operating instructions only serve as a general guideline).
- This single-use biopsy needle is a sterile single-use product intended for use on one patient. Resterilisation and reuse are not permitted. The reuse of single-use products involves a potential risk of infection for the patient or operator. Contaminated products can cause damage to health, disease or even death of the patient. Cleaning, disinfection or sterilisation can adversely affect essential material properties and product parameters and thus result in product failure.
- The product must be disposed of in accordance with hospital regulations and applicable laws.
- Möller Medical GmbH expressly disclaims any liability for damage caused through improper use of the products or use by unauthorised personnel.
- Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

For USA only: When used for breast biopsy, the product is for diagnosis only. The extent of histological abnormality cannot be reliably determined from its mammographic appearance. Therefore, the extent of removal of the imaged evidence of an abnormality does not predict the extent of removal of a histological abnormality; e.g., malignancy. When the sampled abnormality is not histologically benign, it is essential that the tissue margins be examined for completeness of removal of using standard surgical procedures.

ProMag™ 2.2, ProMag™ Ultra and ProMag™ Ultra ST are registered trademarks of Angiotech
 Bard®Magnum® is a registered trademark of C.R. Bard

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